

Instructions for Applying Hoof Wraps Bandages



WWW.HOOFWRAPS.COM

1

Secure the EVA Foam Pad directly to the hoof using Coflex Wrap (Figure 1) OR secure the pad to the center of the bandages using the adhesive backed squares (Figure 2).

Note: In most cases, it is not necessary to trim the pad



Figure 1



Figure 2

2

There are two methods for applying the Hoof Wraps Bandage.

METHOD 1 - Lay bandage flat on the ground. Place hoof on the center of the bandage (Figure 3). You are now ready for Step 3.



Method 1 - Figure 3

METHOD 2 - Pick up the hoof and position the hoof on the center of the pad. Gather the bandage around the hoof without securing "the tabs" (Figure 5). Place the hoof and bandage on the ground and move to Step 3.



Method 2 - Figure 4



TIP - Figure 5

TIP- Pre-attach the rear tabs 1 and 2 (Figure 6) so they form a "cup". Pick up the hoof and place the "cup" on the rear of the hoof and make sure it is positioned in the center of the bandage. Place the hoof and wrap on the ground.

Each Kit Contains:
1 Hoof Wrap Bandage
1 EVA Foam Pad
2 Sets of Adhesive Back Squares

3

Connect tabs 1 and 2 at rear and press firmly. (Figure 6)



Figure 6

4

Connect tabs 3 and 4 in front and press firmly. (Figure 7)

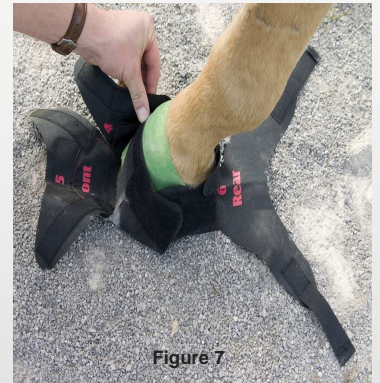


Figure 7

5

Pull front tab number 5 up and over toward the rear side of the bandage and fasten securely. (Figure 8)

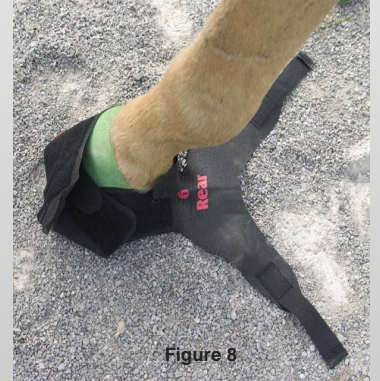


Figure 8

6

Finish by pulling the straps of tab 6 to the front of the hoof and secure firmly. (Figure 9)



Figure 9

Hoof Wraps Can Be Removed, Cleaned And Reapplied